



» ALCOHOL BASED RAPID SURFACE DISINFECTANTS «

ACTOSEPT® AF

Rapid Surface Disinfectant for Medical Devices

- ✓ Contains alcohol 60 % (V/V) and QAC propionate
- ✓ Aldehyde and phenol free
- ✓ Wide material compatibility
- ✓ Bactericidal, mycobactericidal, yeasticidal and virucidal effect
- ✓ Fast drying feature
- ✓ Dermatologically tested and approved by German **DERMATEST** laboratory
- ✓ Certified by the German Society for microbiology and Hygiene **DGHM/VAH**



Actosept® AF is a fast acting ready-to-use surface disinfectant which contains alcohol. Ethanol, 2-propanol and didecyltrimethylammonium propionate containing product can be used for the fast disinfection of small surfaces with its widely effective spectrum and easy use.



German Company



ACTO® Pharma
Cosmetic
Disinfection
Wound Care
Food Supplement
Active Chemical Technology Organisation

ACTOSEPT® AF

Rapid Surface Disinfectant for Medical Device



Areas of Application:

It is used for the fast disinfection of all kinds of surfaces except acrylic glass and alcohol-sensitive surfaces. It is suitable for cleaning and disinfection of medical device surfaces (stethoscopes, cables, connectors, incubators, probes, etc.) that need to be disinfected frequently. It smells fresh and nice.


Method of Use:

Spray the surface to be disinfected from a distance of 30 cm and ensure that the surface is completely covered. Wait 30 seconds for it to take effect. Always complete the disinfection and cleaning process by wiping from clean areas to dirty areas, from farther areas to closer areas, and from top to bottom with a clean and dry cloth. Make sure that the surface is completely wet.


Chemical Composition:

100 g contains:
50 g ethanol, 10 g 2-propanol,
0.05 g didecyltrimethylammonium propionate,
nonionic surfactants and auxiliary substances.


Instructions For Use:

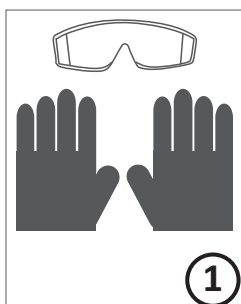


1 Wear suitable protective gloves, goggles and clothing.

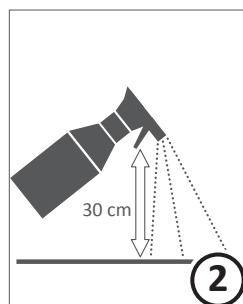

2 Spray the surfaces to be disinfected from a distance of 30 cm and ensure that the surface is completely covered.

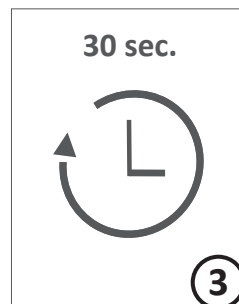

3 Wait 30 seconds for it to take effect

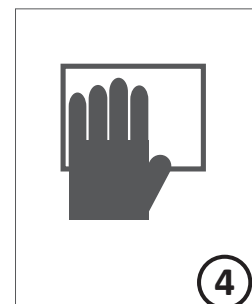

4 Always complete the disinfection and cleaning process by wiping from clean areas to dirty areas, from farther areas to closer areas, and from top to bottom with a clean and dry cloth. Make sure that the surface is completely wet.

Packaging		
Unit	Pieces in a Box	Article-No.
500 ml	18	05.2301.500
750 ml	15	05.2301.750
1 L	12	05.2301.1
4500+500 ml	3	05.2301.45
5 L	4	05.2301.5

Microbiological Activity	Test Method	Concentration	Effect Time
Bactericidal (<i>Inc. MRSA, VRE, A. baumannii</i>)	DGHM, EN 13727, EN 16615	Ready-to-use	30 sec.
Yeasticidal (<i>C. albicans</i>)	DGHM EN 13624, EN 16615	Ready-to-use	30 sec.
Tuberculocidal, Mycobactericidal	EN 14348	Ready-to-use	30 sec.
Limited Virucidal (<i>Corona, Rota</i>)	EN 14476	Ready-to-use	30 sec.



ACTO GmbH
Head Office & Factory
Schulstrasse 8 • 31613 Wietzen-Holte / Germany
Tel: +49 (0) 5022 9801-0 • Fax: +49 (0) 5022 9801-37

Braunschweig Office
Büchnerstrasse 11 • 38118 Braunschweig / Germany

www.actogmbh.com • info@actogmbh.com

